



## Simple Transfers with Active Patients

HandyBelt facilitates short transfers, without mechanical lifting equipment. The belt is used for patients with reduced mobility, but who need only minimal help to perform the transfer themselves. Such situations are, for instance, transfer from sitting to standing, from sitting to sitting, or as support when standing/walking.

The belt is placed on the patient, alternatively on both patient and caregiver, and provides better basis for ergonomically correct work postures compared to purely manual techniques.

HandyBelt is equipped with a strong and secure SafetyBuckle which has been strength tested up to 200 kg (440 lbs.).



Remember that the transfer with HandyBelt should not seem heavy to the caregiver. In that case, mechanical lifting equipment should be used. If the patient has very poor stability or mobility, consider using mechanical aids, such as RollOn or Sabina for raising, or a patient lift with MasterVest or Lift Pants for gait training.

Read the relevant instruction guide for further information on areas of use and practical application. You can also contact your nearest Liko representative for information and advice. You will find our addresses at [www.liko.com](http://www.liko.com).


## PRODUCT DATA

Product Name	Product No.	Size	Maximum Load	Fabric
HandyBelt	3760014	S	200 kg (440 lbs)	Polyester
HandyBelt	3760015	M	200 kg (440 lbs)	Polyester
HandyBelt	3760016	L	200 kg (440 lbs)	Polyester
HandyBelt	3760017	XL	200 kg (440 lbs)	Polyester

HandyBelt can be washed in 60-85° C (140-185°F).

*HandyBelt can be used on both the patient and/or the caregiver to provide extra support during a shorter transfer.*

HandyBelt has 7 to 13 handles, depending on size. The handles give a good grip for the patient or caregiver and are placed both horizontally and vertically along the entire belt. HandyBelt is available in four sizes, from S to XL. Each has a smart colour marking for quick size identification.

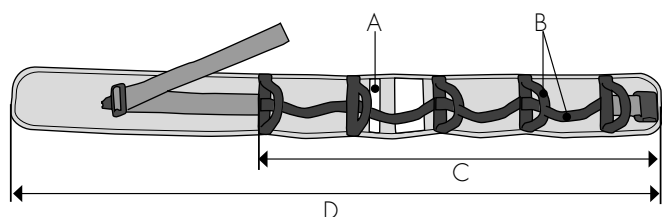

Size Marking (A)	Number of Handles (B)	Length (C/D)
Small Orange 7 60/100 cm (24/40 in.)		
Medium Yellow 9 75/120 cm (30/48 in.)		
Large Blue 11 90/140 cm (36/56 in.)		
Extra Large Black 13 110/180 cm (44/72 in.)		



*The smart colour marking shows the size, and the sturdy handles provide a good grip.*



HandyBelt is a Medical-Technical Class I product.

HandyBelt complies with the requirements for Medical Devices Directive Class I products (MDD 93/42/EEC).

Design and Quality by Liko in Sweden

Liko is quality-certified in accordance with ISO 9001 and its equivalent for the medical device industry, ISO 13485.

Liko is also certified in accordance with environmental standard ISO 14001.



[www.liko.com](http://www.liko.com)

### Manufactured by:

Liko AB  
Nedre vägen 100  
975 92 Luleå  
Sweden  
[info@liko.se](mailto:info@liko.se)

### Local representative in UK

Hill-Rom Limited  
Clinitron House, Ashby Park  
Ashby de la Zouch,  
Leicester LE65 1JG  
Tel: 01530 411000  
Fax: 01530 411555  
[info@liko.co.uk](mailto:info@liko.co.uk)

### To find your local Liko representative,

please visit [www.liko.com](http://www.liko.com) or call  
+46 (0)920 47 47 00



A Hill-Rom Company