



## HandySheet™ and HandyTube™



HandySheet™ and HandyTube™ are manual assistive devices that facilitate transfers by reducing friction between the patient and the underlying surface. They can be used with both patients of normal weight and very heavy patients. The sliding sheets in the Handy series are made of a high-quality, low-friction fabric that slides very easily, even after long-term use.

HandySheet is mainly used to facilitate application of a sling in a seated or recumbent position. HandyTube is used, for example, to reposition a patient higher up in bed, to turn a patient in bed or for transfers between two beds.

Unique features of the HandySheet include pockets that facilitate positioning in a tight-fitting wheelchair, and loops on the longer version of HandySheet, with which the sheet can be secured to head end of the bed. Both features are unique to the Handy series and are registered Liko designs.

# HandySheet



Liko HandySheet is an excellent aid for applying sling, especially on very heavy patients, pain-sensitive patients or patients with contractures.

HandySheet is available in two variants; Short and Long, of which the longer model is equipped with handles all the way around.

HandySheets are delivered in sets of two and are used in pairs.

## PRODUCT DATA

Product name	Prod. No.	Size	Measurements (cm)
HandySheet Short, without handles	3715015	M	70x70
HandySheet Short, without handles	3715017	XL	120x85
HandySheet Long, with handles	3716015	M	90x185
HandySheet Long, with handles	3716017	XL	140x215

Also available in jumbopacks (10 pcs/package).

# HandyTube



Liko HandyTube is a slide sheet that is mainly used to reposition a patient higher up in bed, to turn a patient in bed or for transfers between two beds. HandyTube is available in three different models, each with different application areas: Short, Long and Wide.

## PRODUCT DATA

Product name	Prod. No.	Size	Measurements (cm)
HandyTube Short, without handles	3710015	M	70x90
HandyTube Long, without handles	3711015	M	90x185
HandyTube Wide, with handles	3712015	M	140x140

Also available in jumbopacks (10 pcs/package).

Read the instruction guides for the respective products for more information on application areas and practical use. You can also contact your nearest Liko representative for information and advice. Find our addresses at [www.liko.com](http://www.liko.com). Always remember to work in an ergonomically correct manner. If the task seems heavy for the caregiver, we recommend the use of other lifting and transfer assistive devices in our extensive range of lifts and accessories.



Design and Quality by Liko in Sweden

HandySheet and HandyTube is a medical device class 1 product.

Liko is quality certified in accordance with ISO 9001 and its equivalent for the medical device industry, ISO 13485.

Liko is also certified in accordance with environmental standard ISO 14001.



[www.liko.com](http://www.liko.com)

#### Manufactured by:

Liko AB  
Nedre vägen 100  
975 92 Luleå  
Sweden  
[info@liko.se](mailto:info@liko.se)

#### Liko Distributor in UK:

Liko (UK) Ltd  
Brunel Centre  
Stroudwater Business Park, Stonehouse,  
Gloucester, GL10 3RU  
Tel: +44 (0)1453 827272  
Fax: +44 (0)1453 828844  
[info@liko.co.uk](mailto:info@liko.co.uk)

#### National Distributor in USA and Canada:

Liko  
122 Grove Street  
Franklin, MA 02038  
Tel: 508-553-3993, 888-545-6671  
Fax: 508-528-6642  
[info@liko.com](mailto:info@liko.com)



A **Hill-Rom** Company