




# Solo Octo LiftSheet

English  
7EN160182-01  
2008-10-23


## Product Information

Solo Octo Liftsheet is a disposable lift sheet, intended for use with Liko OctoStretch.

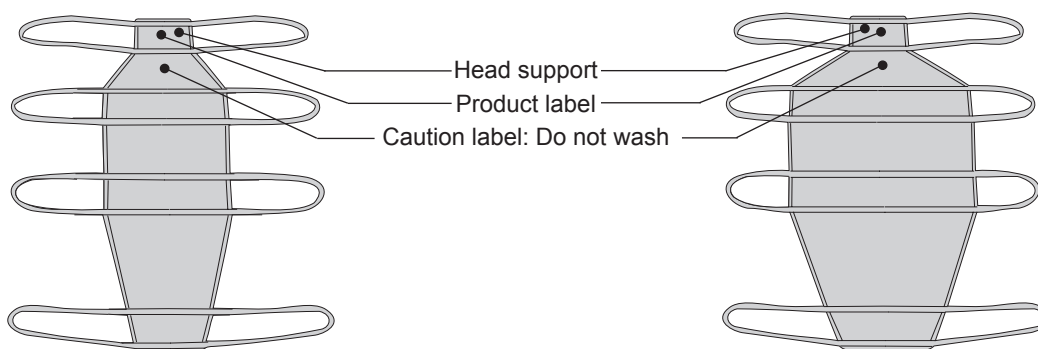

3683905 - Solo Octo Liftsheet, medium  
Sheet size: 500x1600 mm / 19.7x63 inch. (WxL)

3683906 - Solo Octo Liftsheet, large  
Sheet size: 900x1700 mm / 35.4x67 inch. (WxL)

 Carefully study the instruction guide for Liko OctoStretch. For further information about applications and risks, consult the instruction guide for the lift that is being used.

### **IMPORTANT!**

Solo Octo Liftsheet is a disposable item that is intended for individual use only.

When the correct size has been determined for a particular patient, write the patients name on the product label on the back of the lift sheet.

The Solo Octo Liftsheet should no longer be used:

- if it is soiled or if it is suspected of being contaminated
- if it is damaged
- when the patient no longer requires it.

Dispose of the lift sheet in accordance with hospital or care facility legislation.



**DO NOT WASH.** The caution label situated on the back of the lift sheet displays the symbol **DO NOT WASH**. If the lift sheet is washed, this label disintegrates to display the symbol **DO NOT USE**. The lift sheet must then be disposed of.



**Maximum load: 200 kg / 440 lbs.**

 Class I medical-technical produkt



A  Company

[www.liko.com](http://www.liko.com)

#### Manufacturer:

Liko AB  
SE-975 92 Luleå  
Sweden  
[info@liko.se](mailto:info@liko.se)

#### National Distributor in USA and Canada:

Liko North America  
122 Grove Street  
Franklin, MA 02038  
Tel: 508-553-3993, 888-545-6671  
Fax: 508-528-6642  
[info@liko.com](mailto:info@liko.com)

#### Liko Distributor in UK:

Liko (UK) Ltd. • Brunel Centre,  
Stroudwater Business Park,  
Stonehouse, Gloucester  
GL10 3RU, United Kingdom  
Phone: +44 (0) 1453 827272  
Fax: +44 (0) 1453 828844  
[info@liko.co.uk](mailto:info@liko.co.uk)

