

RepoSheet® Saves Backs

Everybody knows that it is almost impossible to lie still in bed – not even when we sleep do we lie still. Even severely disabled persons sometimes end up wrongly positioned in the bed. Consequently, lifting and repositioning patients is one of the most common operations around a bed. Research* also shows that many work-related injuries occur here.

When a patient is to be moved higher up in the bed or turned, RepoSheet is an excellent aid. The RepoSheet is placed under the patient and can be used both in combination with and instead of an ordinary sheet during the entire care period. (Solo RepoSheet is always used together with an ordinary sheet).

With RepoSheet and an overhead lift, it becomes possible for just one caregiver to reposition a patient in bed.




PRODUCT OVERVIEW

Product	Fabric	Size	Prod. No.	Max. Load
RepoSheet Original	Cotton/Polyester	Regular	3687701	200 kg
RepoSheet Original	Polyester HT	Regular	3687301	200 kg
RepoSheet Original	Cotton/Polyester	Ultra	3687702	500 kg
RepoSheet Original	Polyester HT	Ultra	3687302	500 kg

Laundering: 85 °C (185 °F), dry in cabinet dryer or tumble-dry.

Product	Fabric	Size	Prod. No.	Max. Load
Solo RepoSheet 10 pcs	Cotton/Polyester	Short	36871003	200 kg
Solo RepoSheet 2x10 pcs	Cotton/Polyester	Short	36871003-2	200 kg

Laundering:  Non washable. Can be recycled by incinerating.

Contact your Liko representative for advice and information about product changes.

*) *The accident process preceding overexertion back injuries in nursing personnel* by Inga-Lill Engkvist, BSc, Mats Hagberg MD, Eva VVigaeus Hjeltn, PhD, Ewa Menchel, PhD, Lena Ekenvall MD, PROSA study group. Scand J Work Environ Health 1998;24(5):367-375.

A dynamic 3D biomechanical evaluation of the load on the low back during different patient-handling tasks by Skotte JH, Essendrop M, Hansen AF, Schibye B. J Biomech. 2002 Oct;35(10):1357-66.

REPOSHEET® ORIGINAL

RepoSheet Original is available in two versions: Regular, for lifting up to 200 kg, and the sturdier Ultra, which manages up to 500 kg. It is true that transfers can be done by one person only, but we recommend two

caregivers. It is smoother and timesaving as well. RepoSheet Original works best in combination with an overhead lift, for example, Likoall™, Multirall™ or Liko MasterLift™.

Up in Bed



Manually lifting a patient higher up in bed is heavy and strenuous work. It is an operation associated with risks and often repeated many times during the day for each patient.



Whoops, the patient has slid down in bed. It is high time to lift him up with the RepoSheet. The overhead lift is positioned above the patient and the strap loops are attached to the slingbar.



When the patient has been lifted from the bed, he is easily moved higher up with the help of the RepoSheet – all with minimal physical effort.



Liko has a wide range of over-head lifts. Likoall is one of several good alternatives in combination with RepoSheet.



RepoSheet Original is easily placed in the bed. It is soft to lie on and can be used in combination with and instead of an ordinary bottom sheet.

SOLO REPOSHEET®


The Solo RepoSheet is a variant of the RepoSheet, intended for short-term use. The Solo RepoSheet should not be washed, but should be disposed of when soiled or when the patient no longer needs it.



Solo RepoSheet is placed on top of the ordinary bed sheet and can be used during the entire care period. Since laundering is eliminated, the Solo RepoSheet solution is relatively economical, while necessary aids can be held available in storage.



About in Bed

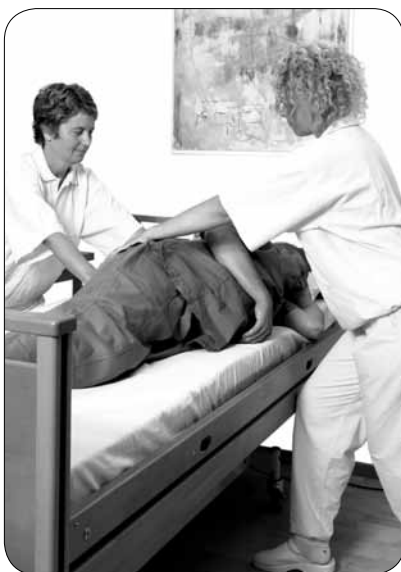

Many patients have to be turned several times a day to avoid, for instance, the risk of bedsores. Even with a well performed manual technique, caregivers are exposed to heavy loads. There is no getting away from this.



First, lift the patient to one side of the bed with the RepoSheet. Then reconfigure the strap loops.



With the help of the overhead lift, turn the patient to the side in a careful and controlled manner.



Support the patient with pillows at suitable places, then detach the RepoSheet Original and place it in the bed.



Using the Solo products from Liko helps to prevent the spread of viruses and contagious bacteria, e.g. MRSA.



www.liko.com

Manufactured by:
Liko AB
Nedre vägen 100
975 92 Luleå
Sweden
info@liko.se

Local representative in UK
Hill-Rom Limited
Clinitron House, Ashby Park
Ashby de la Zouch,
Leicester LE65 1JG
Tel: 01530 411000
Fax: 01530 411555
info@liko.co.uk

To find your local Liko representative,
please visit www.liko.com or call
+46 (0)920 47 47 00

 **Liko**
A  Company

UNDERSTAND, DEVELOP AND ADAPT

Lifting a person requires both good technical aids and knowledge of medical complications. Liko is a specialist in lifting and transferring disabled people. For more than 30 years we have successfully developed and manufactured functional aids which are safe and easy to use. A big part of our success is founded precisely on our combination of high technical competence, a medical know-how, and our own experience in healthcare.

RepoSheet is a good example of Liko's ability to understand, develop and adapt products to different needs within healthcare.

Design and Quality by Liko in Sweden.

RepoSheet is a medical device class 1 product.

Liko is quality certified according to ISO 9001 and its equivalent for the medical device industry, ISO 13485.

Liko is also certified according to environmental standard ISO 14001.

CE



© Copyright Liko R&D AB 2009-10 • Doc. no.: 7EN1 60324-02 • Design: HillRom International Communications

REPOSHEET®



 **Liko**
A  Company