




General Recycling Instructions

3EN800901-03
2010-07-02

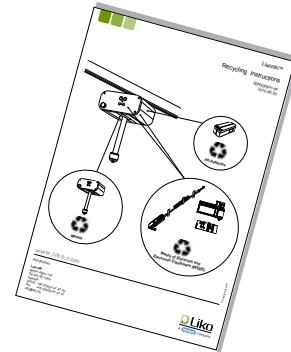

Mobile lifts and overhead lifts

Separate recycling instructions for the different models of mobile lifts and overhead lifts can be downloaded from www.liko.com.

Rail systems and freestanding lift systems

Electrical and electronic components (everything with a electronic card or a cord) should be disassembled and recycled as **Waste of Electrical and Electronic Equipment (WEEE)**.

Other material (rails, upright supports, etc.) should be recycled as **Metals**.

Slings

The entire sling, including laths, padding material, etc. should be sorted as **Combustible waste**.

Furniture Risers

Furniture risers Vivan and Liljan should be sorted as **Combustible waste**.

The plastic guides on furniture riser Viktor should be removed and sorted as **Combustible waste**, and remaining material should be recycled as **Aluminium** or **Metals**.

Other accessories

All electrical and electronic equipment (everything with a circuit card or a cord), for example a hand control, should be disassembled and recycled as **Waste of Electrical and Electronic Equipment (WEEE)**.

Loose components that mainly consist of different kinds of metal (containing more than 90% metal by weight) for example sling bars, should be recycled as **Metals**.

Other textiles or polymers (plastic materials) should be sorted as **Combustible waste**.

www.liko.com

Manufacturer:

Liko AB

Nedre vägen 100
SE-975 92 Luleå
Sweden
phone: +46 (0)920-47 47 00
fax: +46 (0)920-47 47 01
info@liko.se



A **Hill-Rom** Company