


# Viking S / Viking XS

## Quick Reference Guide

English

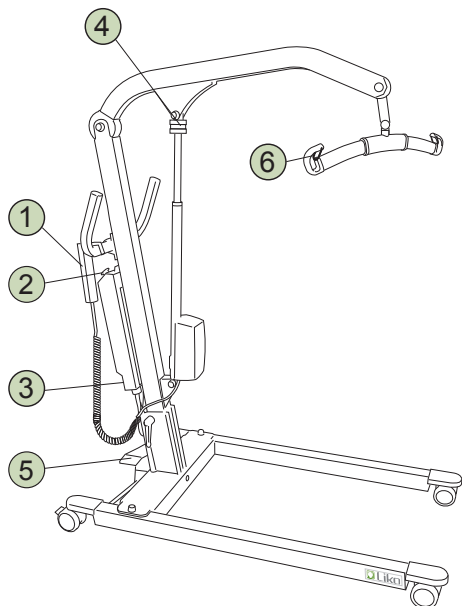

### Safety Instructions


- Read the Instruction Guides for the lift and lifting accessories before use.
  - Plan the lifting operation carefully.
  - Check to ensure that the sling's strap loops are correctly fastened to the slingbar hooks when the sling strap is extended, but before the patient is lifted from the underlying surface.
  - Ensure that the personnel using the equipment have received appropriate instructions and training.
  - **Maximum load: 160 kg (350 lbs.)**
- To retain max. load it is essential to use accessories rated for the same max. load or greater.



### IMPORTANT!

This Quick Reference Guide does not replace the lift unit's Instruction Guide, which can be downloaded from [www.liko.com](http://www.liko.com).


### 1 Handcontrol

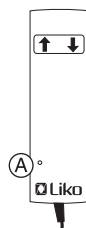

- ⬆ Up
- ⬇ Down
- A Indicating low battery capacity

### 2 Emergency Stop


Activate:

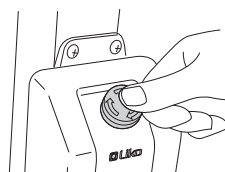


Reset:




### 3 Electrical Emergency Lowering

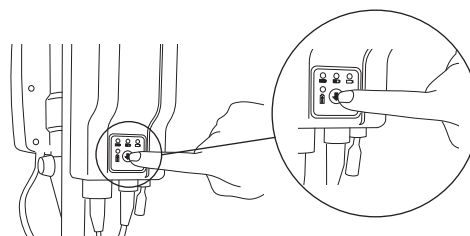


### 4 Mechanical Emergency Lowering

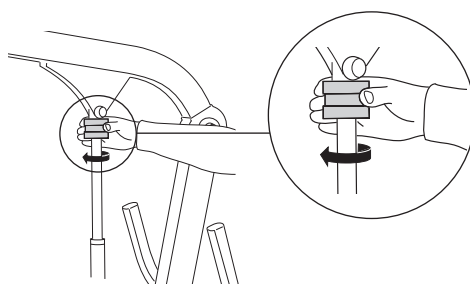


### 5 Base-Width Adjustment

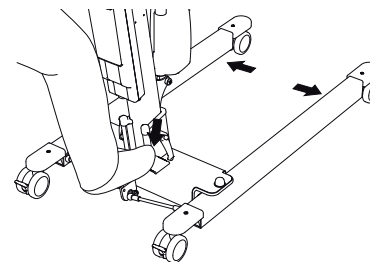

### 6 Safety Latches

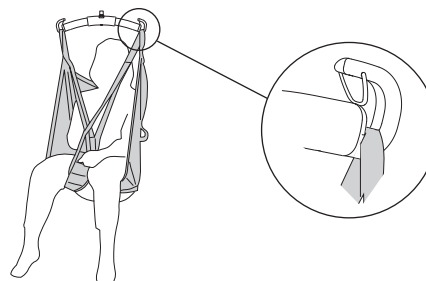


### Charging the Batteries

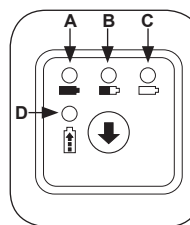

#### Battery Capacity

- A Green: min. 50 %
- B Yellow: 25-50 %
- C Yellow: 0-25 %
- D Charging in progress

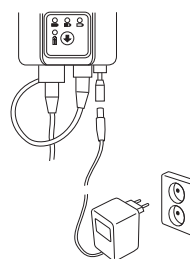

- Charge the lift unit regularly after use.
- The lift unit does not function during charging.
- **The Emergency Stop should not be pressed in during charging.**

### Ergonomics

- Never lift manually – let the lift unit do the lifting.
- Take it easy. Work calmly and methodically.
- Don't strain your back! Work in a walking stance with your back straight.
- Use the bed's raising and lowering functions.
- Reduce the lever effect. Work close to the patient and the lift.

### Cleaning

When necessary, clean the lift unit with common water-based surface disinfectants.

### Simple Troubleshooting

#### Check the following:

- The Emergency Stop is not pressed in.
- The battery is charged.
- The handcontrol is properly connected.

### Other Quick Reference Guides

Quick Reference Guides for Liko's lifts and slings are available for downloading at [www.liko.com](http://www.liko.com).

© Copyright Liko AB



[www.liko.com](http://www.liko.com)

Liko Distributor in UK:

**Liko (UK) Ltd.**  
Phone: +44 (0) 1453 827272  
Fax: +44 (0) 1453 828844  
[info@liko.co.uk](mailto:info@liko.co.uk)

National Distributor in USA and Canada:

**Liko Inc.**  
Phone: +1 508-553-3993, 888-545-6671  
Fax: +1 508-528-6642  
[info@likoinc.com](mailto:info@likoinc.com)